<b>BAUSCH + LOMB</b>	<b>bipolair pinct</b> <b>Bipolaire eraser</b> <b>Intraoculaire bipolaire eraser</b>	<b>NL</b>
----------------------	---	-----------

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## Operating Instructions

#### Precautions

The bipolar forceps/bipolar eraser must be cleaned and autoclaved before they are placed into service the first time and between each use. Use only distilled or deionized water to clean the handpiece. The bipolar forceps/bipolar eraser should be inspected regularly under the microscope for all kinds of damage. Do not use damaged handpieces. Extreme conditions during use and sterilisation may reduce the life/function of this product over a prolonged period of time. The maximum power level should not exceed 7.5W.

#### General Information

Do not clean and autoclave bipolar forceps and bipolar eraser without disconnecting the cord. After sterilisation the Lemo connectors must be completely dry. For additional instructions on the use of this product, refer to the owner’s manual of the corresponding device.

#### Cleaning

Disconnect the bipolar cord from the bipolar forceps/bipolar eraser before cleaning. Always disconnect the Lemo connectors by pulling on the knurled metallic surface, not the cable itself.

- Immediately after each use, all residues should be thoroughly rinsed off the instrument with distilled or deionized water. Use only lint-free and soft cloth. If necessary, use a brush (for example a soft toothbrush) to remove persistent blood, tissue or other debris.

**Caution:** Avoid the use of hard, sharp or pointed objects (scissors, knives, needles, etc.) or highly abrasive sponges for cleaning the handpiece.

- Use a mild detergent solution (neutral pH). Follow the manufacturer’s instructions for administration of the cleaning solution. Rinse thoroughly with distilled water after cleaning to remove all traces of cleaning solution. This prevents instrument discoloration. If balanced salt solution is used, it is extremely important that this solution be thoroughly rinsed off with distilled water, as deposits of chlorid, for example may damage the surface of the instrument.
- Dry the instrument immediately after cleaning by using medical compressed air. Blow drying is the preferred method. Avoid drying the instrument with gauze or cloth, as the instrument may easily be damaged.

#### Sterilisation

Standardised steam sterilisation: 18 minutes at 134°C (fractionated vacuum evaporation). Sterilisation periods of more than 18 minutes do not increase effectiveness.

™/® denotes trademark of Bausch & Lomb Incorporated

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Bausch & Lomb GmbH